

# Style Italiano Colour Recipes

3M™ ESPE™ Filtek™ Ultimate Universal Restorative



## Espertise™

# Learning

### Creating Aesthetic Restorations: A Simple Dual-Shade Technique

#### The Recommended Recipes:

To get	use this	with this
<b>A0</b>	Dentin <b>W</b> + Enamel <b>A2</b>	
<b>A1</b>	Dentin <b>A1</b> + Enamel <b>A3</b>	
<b>A2</b>	Dentin <b>A2</b> + Enamel <b>A3</b>	
<b>A3</b>	Dentin <b>A3</b> + Enamel <b>A3</b>	
<b>A3.5</b>	Dentin <b>A4</b> + Enamel <b>A3</b>	
<b>A4</b>	Dentin <b>A4</b> + Body <b>A4</b>	



#### Additional characterizations can be created using:

- Clear Translucent (CT)
- Enamel Shade:  
Extra White (XW)
- Body Shade: A6