IMPRIMO® LC Splint Produktinformation

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IMPRIMO® LC Splint Product Information

Description:

IMPRIMO® LC Splint is a light-curing resin especially engineered on the base of methacrylate for high-precision occlusal splints and drilling templates.

Hints for processing/use:

Material should be filled up to the maximal level mark of the IMPRIMO® Tray, wearing your personal protective equipment (goggles, gloves and mask). Make sure to remove small air bubbles using a spatula.

Post-processing:

Make sure to remove uncured material residues from the polymerised model using Isopropanol. After cleansing place the completely clean models in a bowl filled with water for final curing. Place the bowl in a light-oven (e.g. LC-6 Light Oven) for 5 minutes. Thanks to the water, direct contact between oxygen and light-curing resin and consequently the formation of an inhibition layer will be avoided. Finally, the surface of the polymerised splints can be high-gloss polished.

Any changes of the described fabrication process may lead to modifications of the mechanical properties or material colour.

In case of material impurities like dirt or debris, the material can be filtrated and repeatedly mixed thanks to its low viscosity. Allow material to rest for approx. 1 hour in order to avoid air inclusions.

Storage:

Material should be stored at room temperature (18° to 28°C) and protected from light. Light might damage the material.

Hazard statements:

Avoid lengthy skin contact with un-polymerised material and inhalation of monomer vapor. In some isolated cases contact with single components of IMPRIMO® LC Splint may cause allergic reactions. For further details please refer to the corresponding safety data sheet of IMPRIMO® LC Splint. In case of accidental skin or eye contact make sure to rinse thoroughly, consult a doctor, if necessary.

Attention: Polymerised resins are chemically persistent. Make sure to avoid clothing stains.

All information on processing of our materials- verbal, written or practical, is given to the best of our knowledge and to be understood as hints. Material application and processing is beyond our control and lies within the responsibility of the user.

To be used only by authorized staff.

Material Safety Data Sheet: www.scheu-dental.com/downloads